



easyforce

P E R F O R M A N C E


Manual

EasyForce Performance dynamometer measures peak force capacity of 300 Kg/2942 N which enables testing compound movements with higher peak force production. This pull-type dynamometer is suitable for strength and performance coaches, sports physiotherapists, personal trainers and other movement professionals striving to improve strength assessment. If your daily work revolves around athletic population returning to sports and performance, amateur athletes and individuals wanting to improve their strength based on data – EasyForce Performance is the right product for you.

Together with ISOplate assessment, tests like Isometric Thigh Pull, Isometric Deadlift, or Isometric Pushup are performed safer than ever, and in a blink of an eye. Track peak force, time-to-peak force and average force without hassle and worry about complicated data exports or connectivity. Accurate, Simple and Quick access to data that matters for you and your clients!

CONTENTS OF THE BOX




- 1 x EasyForce unit [1]
- 1 x USB cable [2]
- 2 x Carabiners [3]
- 1 x Manual [4]

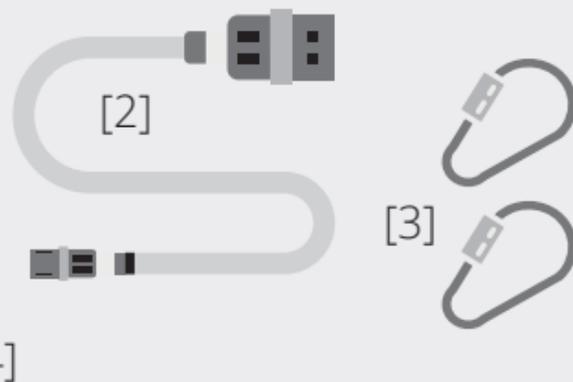

Selecting the measurement unit




History view and deleting the value

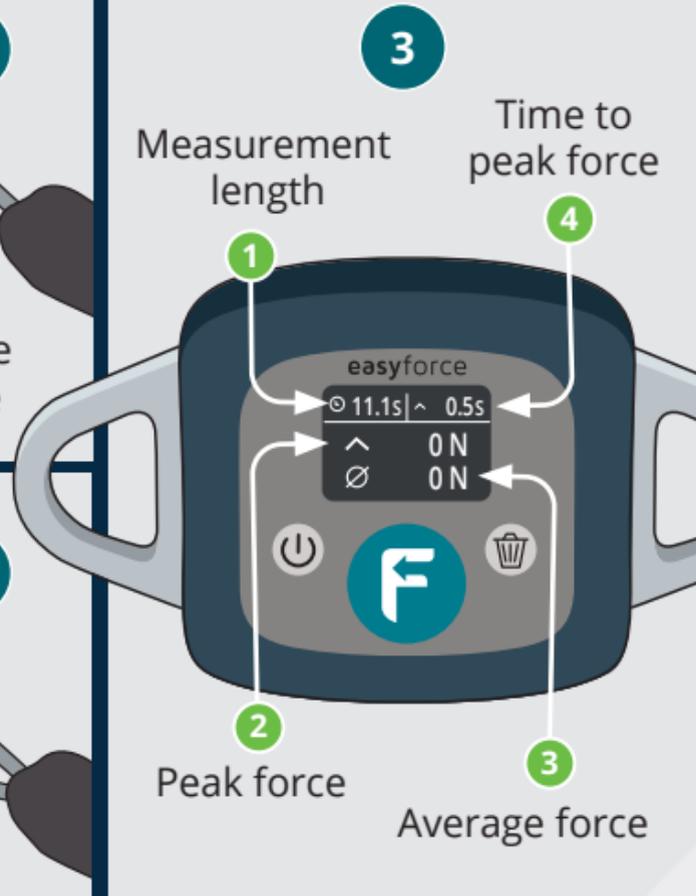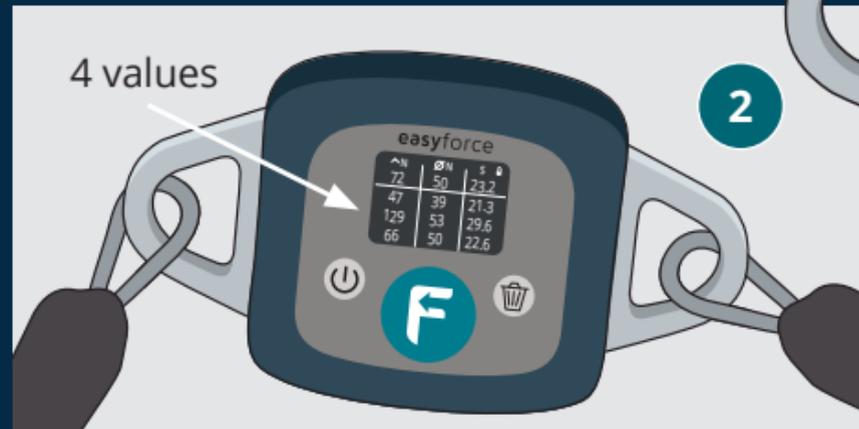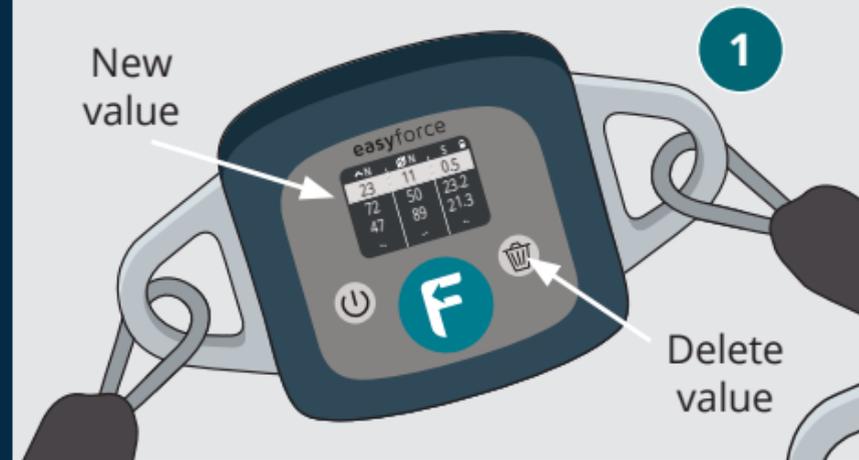

Newest measurement on top. The 4 last values are saved.

Start and end
the measurement

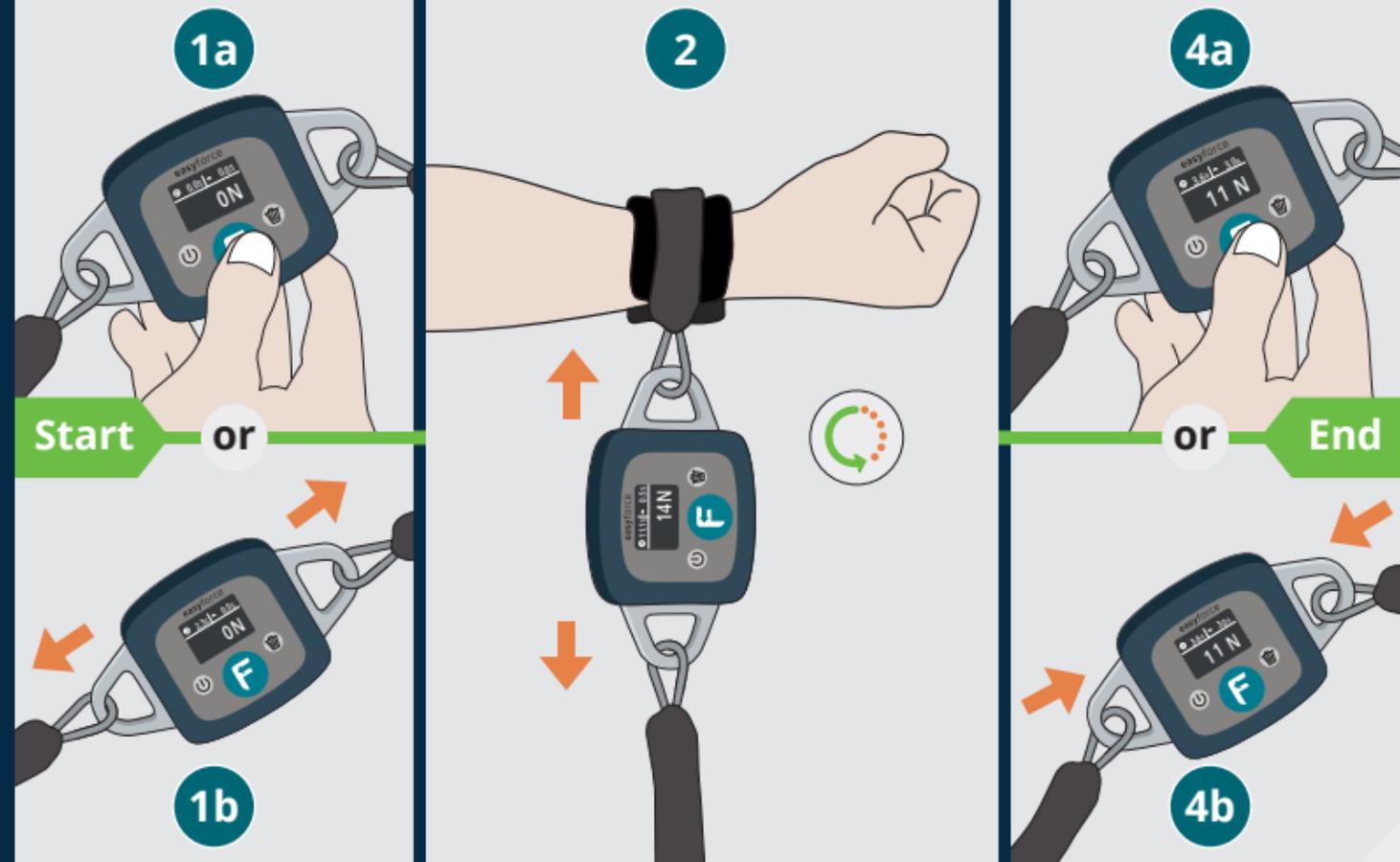

Start with 1a or 1b and end with 4a or 4b.

Parameters



Peak Force

- maximum force recorded

Average force

- average of force samples across the whole measurement time


Time to peak force

- time it took for the client to reach peak force

Time under tension

- total measurement time

Measurement length [time under tension]

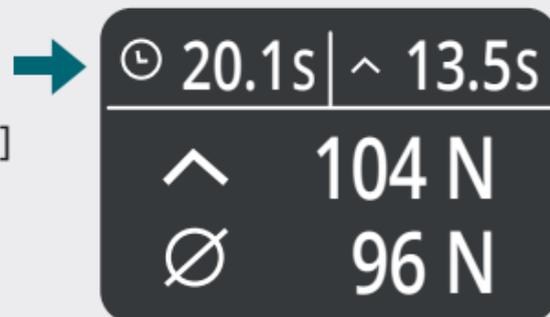

Product specifications



Product Name	EasyForce Performance
Operating Environment	Not for home use
Operating Temperature	10°C/50F to 45°C/113F
Storage Temperature	0°C/32F to 45°C/113F
Enclosure	IP4X
Operating Time	3 weeks (at 12 min/day)
Maximum rated force	300 kg
Stand-by time/charging time	15 weeks/5 hours
Accuracy	99 %
Charger specifications	5VDC, >0.5A, USB Micro

Register your product for an additional year of warranty

Register your EasyForce for an additional year of warranty (two years instead of one), as well as access to the latest instruction films and clinical studies. Registration is free and only requires name, e-mail and the serial number of your EasyForce.



Register at
meloqdevices.com/register

Instructional videos and posters



Download or order an **instructional poster** for **EasyForce Performance** here

meloqdevices.com/pages/easyforce-performance-dynamometer



EasyForce Performance
instructional videos on **YouTube**
youtube.com/c/Meloq



Follow us on social media!

Please, use the hashtags #measuremore #measureforsuccess

[instagram.com/meloq_devices/](https://www.instagram.com/meloq_devices/)

[facebook.com/MeloqDevices](https://www.facebook.com/MeloqDevices)

[linkedin.com/company/meloq-ab/](https://www.linkedin.com/company/meloq-ab/)



Manufactured by Meloq AB

• contact@meloq.se • meloqdevices.com

